

Extension Cord

BM-918B

General

The BM-918B extension cord extends the multi-link cable of the AE-918P neuro unit by 1.5 meters.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Connect the extension cord only to the specified instrument and follow the specified procedure. Failure to follow this warning may result in electrical shock or injury to the patient and operator, and cause damage to the instrument leading to inaccurate monitoring.

⚠ WARNING

Do not put the extension cord on a floor. If the cord gets dusty, damaged or contacts liquid, it may cause malfunction on the connected device and electrical shock to the patient and operator.

⚠ CAUTION

- Do not pull or bend the extension cord with too much force. This may cause damage on the cord or incorrect measurement.
- Hold the connector of the extension cord when connecting or disconnecting the extension cord.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599