

Cart

KD-033A, KD-033AK

General

The KD-033A/AK cart is for mounting an MEE-2000 neural function measuring system.

Safety Information

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

Only use this cart and LCD mount for an MEE-2000 system. Do not mount any other instruments which are not specified. The cart may tip over or the devices may fall off and cause injury.

⚠ CAUTION

Use the specified tool for cart assembly.

⚠ CAUTION

Be careful not to pinch your fingers between the components and cart.

⚠ CAUTION

Make sure that the bracket is attached to the stopper. Otherwise, the LCD mount may fall off the stopper.

⚠ CAUTION

To prevent the cart from tipping over or the devices falling off the cart:

- Do not put or hook anything on the handle or pole.
- Do not ride on the cart.
- Do not lean on the handle or put your weight on the cart.
- Make sure that the cart is on a flat surface which is not sloped.
- Always lock the casters so that the cart does not move accidentally.

⚠ CAUTION

- Release the caster locks when moving the cart. Do not move the cart with the casters locked. The cart may tip over.
- When moving the cart, hold the handle.

⚠ CAUTION

Be careful not to pinch your fingers when you adjust the LCD mount.

⚠ CAUTION

Do not put heavy objects in the basket. The cart may tip over and the basket may fall off.

⚠ CAUTION

Periodically check that the casters rotate smoothly and that no screw or knob screw is loose. Otherwise the cart may tip over or the devices may fall off.

⚠ CAUTION

Securely install the amp unit, breakout box, stimulation pod and TcMEP switch box on a rail. If the instrument falls off, the patient or operator may get injured or the instrument may be damaged.

⚠ CAUTION

Only use the provided or specified screws. Make sure that the screws are securely tightened. If screws are loose, the components may fall off and the patient or operator may get injured.

⚠ CAUTION

When moving the system, select a flat path and move the cart gently. Otherwise, the cart may tip over and cause injury.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 11 Jul 2019
4th Edition: 27 Feb 2023

