

# Water Trap (Adult, Neonate)

## YG-300P, YG-301P

### General

The YG-300P water trap (for adult) and YG-301P water trap (for neonate) are used with a GF-300 series multigas unit.

Each water trap uses a hydrophobic filter to separate water droplets, secretions, bacteria and dust from the measured gas and stores the separated substances temporarily in the container section of the water trap, preventing them from entering the interior of the multigas unit.

### Safety Information

**⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

**⚠ CAUTION**

Insert the water trap so that its direction matches the direction of the installed unit. If the water trap is inserted in the wrong direction, the maximum performance from the water trap cannot be guaranteed and the unit may get damaged.

**⚠ CAUTION**

- Write the starting date of the water trap on the front of the water trap with an oil based marker. Refer to this date when replacing the water trap with a new one.
- Replace the water trap every month or when the "CHANGE WATERTRAP" message appears on the monitor screen. If the water trap is not replaced, the hydrophobicity of the filter is reduced, water penetration cannot be stopped, the unit may be damaged and measurement accuracy cannot be guaranteed.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>

**EC REP** European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffeisenstrasse 10, D-61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599