

Nurse Call Cable

YJ-672P

General

The YJ-672P nurse call cable connects a BSM-3000 series or BSM-6000A/K series bedside monitor to a nurse call system. Refer to the bedside monitor and nurse call system manuals together with this manual.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Do not rely on only the alarm information from the nurse call system to determine the patient condition. Confirm the waveforms, measurement data and alarm information on the bedside monitor. Failure to consider other patient data may cause a misunderstanding of the patient condition.

⚠ CAUTION

Alarm and parameter information might not be sent to the external instruments in the following situations.

- Communication failure due to a damaged connection cable or connector or poor contact between the bedside monitor and nurse call system
- Nurse call system failure

⚠ CAUTION

Cut the unused signal lead wire. Otherwise, it short-circuits to other lead wire pins and damages the monitor.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599